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REHASHOP Vollgummirampe Gecko

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Instruction Manual

REHASHOP solid rubber threshold ramp Gecko

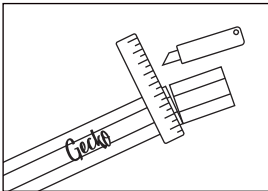
Thank you very much for your confidence in our products.

Please read the following information and instructions carefully before using your new REHASHOP solid rubber threshold ramp Gecko.

If you have any questions, please contact our customer service team. You can reach them free of charge at the number **0800 888 90 80**.

We wish you all the best with your new product!

Your rehashop team

You can easily cut your solid rubber threshold ramp Gecko to the right length using a ruler and a cutter knife or scissors.

Information:

- Included in delivery are the REHASHOP solid rubber threshold ramp Gecko and double-sided adhesive tape.
- The REHASHOP solid rubber threshold ramp Gecko is suitable for indoor and outdoor usage. Outdoor weather conditions can cause wear over time.
- Use the supplied adhesive tape in the interior. It is particularly suitable for smooth surfaces, such as parquet or linoleum.
- The ramp has a ribbed underside. It adheres to rough surfaces even without adhesive tape.
- Please replace the solid rubber threshold ramp if the adhesion is no longer ensured.
- Usage: Measure the door width before using your new ramp. With the help of a ruler and a knife, you can easily cut the appropriate length. Caution: Always be careful when handling sharp objects, such as scissors and knives.
- Cleaning: The REHASHOP solid rubber threshold ramp Gecko can be cleaned with common household cleaning agents.

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