Elektromobil Lion

Gebrauchsanweisung | Mode d'emploi
Instruction manual

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Thank you for your trust in our product!

To ensure safe handling of your new electric vehicle, pay particular attention to the following usage and safety instructions.

If you have any questions, please contact our customer service team. You can reach our service staff on the free hotline for the respective country.

Germany:	+49 (0)6421 - 30 30 800
Austria:	+43 (0)1212 - 62 84
Switzerland:	+41 (0)32 - 372 7550
Switzerland/Romandy:	+41 (0)32 - 372 7550
France:	+33 (0)277 - 41 20 96

We hope you enjoy your new electric vehicle!

Your rehashop team

These instructions for use have been prepared based on the best information currently available and translated from German. Because product details are subject to change, the product purchased may not exactly match the description in this guide. The latest instructions for use can be requested in the online shop. This is revised regularly to keep you up to date with new information.

- Before using the electric vehicle for the first time, be sure to read these instructions for use carefully and follow the safety instructions to ensure proper use.
- Keep these instructions for use for later reference and pass them on to other users.
- Any liability is excluded if the safety rules stated in the instructions for use are not observed.
- Failure to follow these instructions for use and/or safety precautions may result in falls and serious injuries

CONTENTS

	Page
1. IMPORTANT INSTRUCTIONS	48
1.1 Explanation / symbols in the instructions for use	
1.2 Importer Germany / Switzerland	
2. INSTRUCTIONS FOR USE	49
2.1 Introduction	
2.2 General information	
2.3 Intended Use	
2.4 Safety instructions	
3. PRODUCT DESCRIPTION	52
3.1 Scope of delivery	
3.2 Electric vehicle overview	
3.3 Product dimensions and specification (electric vehicle)	
4. HANDLING	54
4.1 Armrest	
4.2 Seat adjustment	
4.3 Steering column position	
4.4 Control panel and its function	
4.5 LCD-Screen	
4.6 Control rocker forward	
4.7 Control rocker backwards	
4.8 Brake	
4.9 Freewheel lever	
4.10 Turn ON / Turn OFF	
4.11 Tires	
4.12 Function during strong temperature fluctuations	
5. BATTERY AND CHARGER UNIT	58
5.1 Application and security measures	
5.2 Product dimensions and specification (charger)	
6. TROUBLESHOOTING / FAULT RECTIFICATION	61
7. SYSTEM DIAGNOSIS	62
8. STORAGE	63
9. CLEANING	63
10. MAINTENANCE	63
11. ELECTROMAGNETIC INTERFERENCE (EMI)	65
12. GUARANTEE	65
13. ATTENTION!	66

GB

1. IMPORTANT INSTRUCTIONS

1.1 Explanation / symbols in the instructions for use

Follow the instructions!

Manufacturer

Order number

Swiss importer

Batch name

European importer

Instructions for use contain important safety-related information

The Lion electric vehicle complies with Regulation (EU) 2017/745

This device does NOT belong in household waste

Allergy sufferers note: Control elements such as handles, switches, buttons as well as tires, seat and armrests may contain natural rubber latex. Natural rubber latex can cause allergic reactions including anaphylactic shock.

1.2 Importer

Germany

PROTENO GmbH

Niederwetttersche Str.1
35094 Lahntal
Germany

Tel: +49(0)6421-3030800
E-Mail: info@rehashop.de
Web: www.rehashop.de

MedPath GmbH

Mies-van-der-RoheStraße 8
80807 Munich
Germany

Tel: +49(0)89-189174474
E-Mail: info@medpath.pro
Web: www.medpath.de

2. INSTRUCTIONS FOR USE

2.1 Introduction

The electric vehicle is used to transport a person while sitting. The armrest, which can be raised completely, makes getting in comfortable. The maximum speed is 6 km/h with a climbing ability of up to 10°. With the 500 watt rear wheel motor drive, the electric vehicle achieves a range of approx. 30km* when fully charged.

*Range measured under standardized test conditions, actual range may vary.

2.2 General information

These operating instructions are intended to help you familiarize yourself with the operation and handling of the electric vehicle. Keep the instructions for use within easy reach and give them along when you pass on the mobility scooter. Please read the instructions for use completely before taking your first ride.

GB

2.3 Intended use

The electric vehicle is intended exclusively to increase mobility and transport one person. The electric vehicle is designed for a maximum load of 160 kg. A guarantee can only be given if the product is used under the specified conditions and for the intended purposes.

2.4 Safety instructions

Only use the mobility scooter if it is in perfect condition. Improper use of the mobility scooter can put the user at risk.

FOR YOUR SAFETY, YOU SHOULD REMEMBER THE FOLLOWING:

SAFETY INSTRUCTIONS

- Always have defects repaired immediately by a qualified dealer.
- Do not make any unauthorized changes to the electric vehicle. contact contact your dealer for this.
- Improper modifications may result in malfunction and increased lead to risk of accidents.
- Prolonged contact with water or high humidity can cause the metal to rust, so dry the mobility scooter carefully.
- Please note that upholstered parts that are exposed to direct sunlight heat up and can cause skin injuries if touched. Therefore, cover these parts and protect the mobility scooter from direct sunlight.

SAFETY INSTRUCTIONS

- The product has not been tested according to DIN 1021-1 and 1021-2 and is therefore not considered flame retardant. Protect your mobility scooter from sources of fire such as cigarettes, lighters or open flames.
- As an electric vehicle driver, you are part of public road traffic and we would therefore like to point out that you are therefore subject to the road traffic regulations.
- Before using the mobility scooter for the first time, have it checked by a specialist schools.
- Before each use of the electric vehicle, please check whether all attached parts are properly attached.
- Drive particularly carefully on your first trip and get used to the effects of shifts in the center of gravity and the driving behavior of the electric vehicle.
- To avoid falls, practice riding on level, manageable terrain. A companion is recommended.
- Please note the stated maximum payload of 160 kg. Also take into account additional weight, such as the bag and luggage.
- The electromagnetic brake secures the electric vehicle against unintentionally rolling away when stationary.
- If possible, keep your hands and fingers away from all moving parts. Be aware of the risk of jamming with moving parts.
- Never use the mobility scooter on an escalator. If access ramps are available, they must be used.
- The electric vehicle is designed to transport one person only. One any other use is considered improper.
- Before every trip, check the drive wheels, all cable connections and electrical connections, the seat and backrest, the tires and the brakes.
- Please note that when driving on slopes or on wet, slippery surfaces, the braking effect of the tires is significantly reduced. Always adapt your driving style to your surroundings.
- In other means of transport (e.g. bus or train), the user is not allowed to sit in the electric vehicle while driving. Only the seats permanently installed in the respective means of transport offer sufficient protection in dangerous situations.
- This mobility scooter is not a replacement for a car seat. The electric vehicle user must always move to a designated seat for transport. When transporting the electric vehicle, only attach it to designated places.
- When driving on a slope, it must be noted that there is a risk of tipping over 10%. If you are climbing a step or an incline forward, your body should be tilted forward.
- The electric vehicle may only be driven on solid surfaces. There is an increased risk of tipping over on poor roads (gravel, potholes, curbs).

SAFETY INSTRUCTIONS

GB

- Avoid shifting your weight that could cause you to tip over. This happens through one-sided strain, a change in posture or attaching a bag to the mobility scooter.
- The mobility scooter is not intended to be used as a support for standing up from a squat or from a position sitting intended.
- The wheels can get stuck on uneven ground, for example in gutters or gratings. Therefore, avoid this type of obstacle as much as possible.
- Always load the seat vertically from above!
- There is a significant risk of slipping on wet, dirty, icy or snow-covered surfaces. The braking effect is severely limited on these surfaces.
- Wear light-colored clothing when it is dark. If possible with reflective goals.
- Never stand on the mobility scooter!
- DO NOT use music entertainment systems while driving to avoid accidents caused by distractions.
- Only use the mobility scooter on sidewalks - The mobility scooter is NOT to be equated with motorized vehicles such as cars or motorcycles and therefore cannot be used on the street.
- If the mobility scooter malfunctions while crossing the street, immediately call passers-by for help. Set the scooter to manual mode using the freewheel lever and let the scooter push to continue your journey.
- Please keep your mobility scooter at a safe distance from pedestrians and other objects.
- Be careful of steep slopes! The gradient should not be more than 10 degrees and slow down the speed.
- Avoid driving sideways on slopes. Attention: danger of tipping over!
- Some of the components may potentially cause damage if they come into contact with other objects.
- DO NOT remove the anti-tip wheel if the mobility scooter is equipped with one.
- To avoid electromagnetic interference, stay away from strong. Keep away from magnetic fields and current inductors.
- Driving an electric vehicle while under the influence of alcohol is strictly prohibited.
- If you feel dizzy or your vision is blurred due to health problems or medication, do not operate the mobility scooter.
- Consult your doctor if you are taking prescribed medications or if you have certain physical limitations. Some medications and restrictions may affect your ability to operate the mobility scooter in a safe.
- If you have any problems or questions, please contact your specialist dealer.

3. PRODUCT DESCRIPTION

The electric vehicle is delivered in a box. After unpacking, keep the box and packaging material for later storage if possible. Check the delivery contents for completeness and integrity. If there is any damage, do not use the product. First inform your specialist dealer.

3.1 Scope of delivery

The scope of delivery of the "Lion" electric vehicle includes:

- 1 Electric vehicle
- 1 Instruction manual
- 1 Seat (adjustable)
- 2 Batteries
- 1 Charger
- 1 Basket

3.2 Electric vehicle overview

- | | | |
|------------------------------|---|---|
| 1. Side mirror | | |
| 2. Control element (control) |  | |
| 3. Indicator | | |
| 4. Cup holder | | |
| 5. Basket | | |
| 6. Front light | | |
| 7. Footrest |  | |
| | 8. Headrest (height adjustable) | |
| | 9. Backrest (adjustable) | |
| | 10. Armrest (foldable) |  |
| | 11. Backrest lever | |
| | 12. Comfort seat | |
| | 13. Tires |  |
| | 14. Anti-Tip protection | |

3.3 Product dimensions and specification (Electric vehicle)

Model	SW1260-S25
Total length	1200mm
Total width	620mm
Total height	1230mm
Weight including battery	86kg
Max. Load	160kg/1Person
Speed	6km/h (forward)
Turning circle	1650mm
Max. incline and decline	10°
Max. range	30 ± 5km* (reference measurement)
Drive system	Rear wheel drive
Brake system	Intelligent electromagnetic brake
Motor	24V/500W/350W
Control unit	24V/90A
wheel size	10" 3,5"* (reference measurement)
Battery	24V/38AH (2pieces) lead-acid
Charger	DC24V4A AC115-230V
Max. charging current	4A
Charging time: up to 80% charge	approx. 8h
Charging time: up to 100% charge	approx. 10h

(The information provided above is for reference only.)

4. HANDLING

4.1 Armrest

The electric vehicle is equipped with swiveling armrests, which makes getting in and out easier. For adjustment you will find an adjustment wheel on the armrest.

4.2 Seat adjustment

The angle of the seat back is adjusted to the desired position using the lever on the seat.

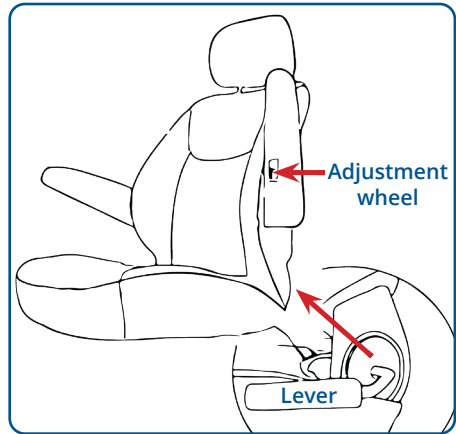
ATTENTION:

There is a risk of jamming when the armrest is swiveled upwards. Always move the armrest slowly and pay attention to your fingers.

ATTENTION:

The armrests are not suitable for pushing or lifting the mobility scooter!

4.3 Steering column position

To adjust the steering column, press the lever down and position it at the desired angle.

4.4 Control panel and its function

1. Left indicator
2. Headlight
3. Horn
4. Right indicator
5. Max speed regulator
6. Warning signal light
7. Horn

Max speed regulator

You change the maximum top speed by turning the control to the left for low top speed or to the right for maximum top speed.

Headlight

In poor visibility conditions such as fog, dusk and darkness, switch on the headlights. Pressing the button again switches the light off again.

Horn

Press the horn in dangerous situations or to draw the attention of pedestrians and vehicles.

Indicator

To initiate a turn, press the left or right turn signal button. You can then turn in the direction you want.

4.5 LCD-Screen

1. Turn signal indicator on the left
2. Outside temperature display
3. Display of the speed level (H)high/(L)low
4. Forward display/reverse display
5. Battery charge level display
6. Turn signal indicator on the right
7. Manufacturer symbol
8. Display in the event of a fault (READY symbol flashes)
9. Speed display in km/h
10. Daily mileage display (max. 99.9 km)
11. Display of the running time used per usage

LONG DISTANCES:

For long distance travel, the battery should be fully charged. If only one or two bars can be seen on the battery charge level display under point 5, there is only a small remaining range. Recharge the battery as soon as possible.

4.6 Control rocker forward

To move forward, pull the right lever towards you or push the left lever away from you.

4.7 Control rocker backwards

If you want to reverse, pull the left lever towards you or push the right lever away from you.

ATTENTION:

A warning signal sounds when reversing. Please be aware of your surroundings.

4.8 Brake

To brake, release the control lever. The electric vehicle brakes automatically thanks to the intelligent engine brake. When no drive lever is operated, the intelligent electromagnetic braking system also acts as a parking brake.

ATTENTION:

When going downhill, the brakes apply automatically to avoid exceeding the maximum speed.

GB

4.9 Freewheel lever

For coupling / decoupling engine and transmission. To drive, the engine and transmission must be coupled.

ATTENTION:

Please note that the electric vehicle is not braked if you decouple the motor and gearbox using the freewheel lever. In this case, the automatic braking system only kicks in when you reach 30% of the maximum speed.

4.10 Turn ON / Turn OFF

Turn the key to the right to turn on the mobility scooter. The LCD screen lights up and the mobility scooter is ready to go. To turn it off, turn the key to the left.

ATTENTION:

Before getting on or off, it is necessary to ensure that the key switch is in the "OFF" position.

4.11 Tires

The electric vehicle is equipped with pneumatic tires, so the tire pressure should be checked once a week. The correct tire pressure extends the service life and ensures a comfortable ride. Make sure the tire pressure is between 2 - 2.4bar.

ATTENTION:

Use a suitable air pump and inflate the tire carefully to prevent bursting. Pay attention to the recommended tire pressure.

4.12 Function during strong temperature fluctuations

The electric vehicle is sensitive to strong temperature fluctuations. Safe operation is only guaranteed within a temperature range of -25°C to +50°C.

ATTENTION:

At high temperatures, lower speeds may be used to prevent overheating and potential damage to the engine. This can lead to lower performance as the chemical processes in the batteries can slow down in extreme temperatures, resulting in lower performance.

ATTENTION:

At very low temperatures, the components of an electric motor can freeze, which can cause the motor to not start or reduce its performance.

5. BATTERY AND CHARGER UNIT

Before first use, you should read all instructions carefully to avoid damage or injury. Charging outdoors, in confined spaces or in areas exposed to intense sunlight, or near flammable sources such as gas, oil mist or steam is not permitted.

5.1 Application and security measures

Before connecting the charger, the mobility scooter must be switched off. Only use the charger provided. Before plugging in the power plug, remove any jewelry as this may cause a short circuit. Check the charging plug for dryness and damage before connecting it and push the cover to the side. Charge the mobility scooter in a room with good ventilation and on a stable surface. Avoid extreme heat or direct sunlight as well as temperatures below -25°C. Charging time is approximately 10 hours. After completing the charging process, immediately disconnect the charger from the mobility scooter and the socket.

IMPORTANT:

- A new battery only reaches full performance after 2-3 charging and discharging cycles
- After one month of non-use, the battery loses approximately 5% capacity.
- A charging cover prevents moisture from getting wet, so close the cover after use and store the charger in a dust-free and dry environment between -25°C and +40°C.
- Do not interrupt charging until the process is complete.
- The electric vehicle is equipped with protection against overcharging.
- If you do not use the mobility scooter for a long period of time, the battery should be stored separately from the charging unit.
- Store the battery with a full charge and check the charge level once a month. Make sure that the battery does not discharge completely and, if necessary, fully charge it again.
- The mobility scooter has a RESET button that is automatically triggered in the event of a malfunction. To continue charging, turn off the charger, press the RESET button and turn on the charger again.

GB

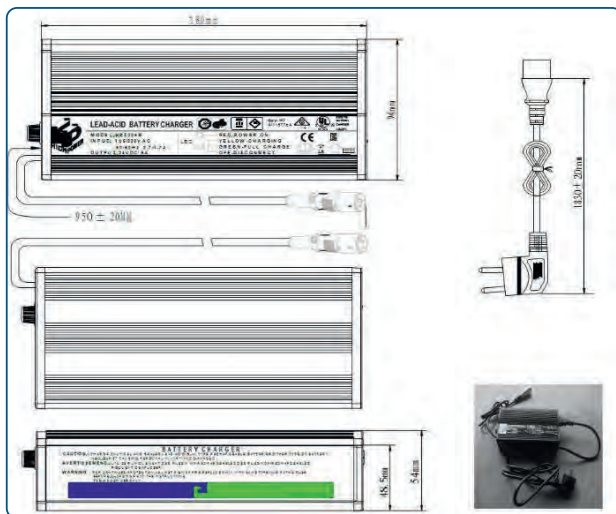
ATTENTION:

- If you notice any defects, cracks or swelling on the surface of the charger, DO NOT use the charger and inform your dealer immediately. Repairs may only be carried out by an authorized dealer
- Batteries do NOT belong in household waste. For proper disposal, please contact the public collection points in your community.
- Do not unplug the power cord when the outlet or your hands are wet, as this may cause electric shock.
- The battery is flammable, please be careful not to smoke near the battery, especially when it is charging.

TIP:

- If you drive the mobility scooter with a low battery level and have no opportunity to charge it in time, the range can be increased if you set the speed to the lowest level (see point 4.3).

5.2 Product dimensions and specification (charger)

Model	HP8204B
Efficiency	81% min.
Output current	5A $\pm$ 0,2A max.
Output voltage	29,8 V $\pm$ 0,2 V max.
Maintenance voltage	31,2 V $\pm$ 0,2 V
Input current	2,7 A max.
Input voltage	115 VAC/230 VAC 50/60 Hz
Pulsating stream	0 ~ 5A
Protection	1. Over-current protection 2. Over-voltage protection 3. Reverse current protection 4. Short circuit protection
LED Display	red: On orange: Loading green: Fully charged
Operating temperatur	0°C ~ 40°C
Operating humidity	20% ~ 85%
Operating altitude	0 ~ 2000m
Dimensions	180mm $\times$ 93mm $\times$ 53mm
Weight	0,83 kg
Color	black

The charger can operate on 230 V or 110 V. Use a small screwdriver on the red controller to make the desired setting. The default setting is 230 V.

6. TROUBLESHOOTING / FAULT RECTIFICATION

Mistake	Possible Cause	Solution
Range small	Battery not fully charged	Fully charge battery
	Battery capacity has decreased (e.g. due to age)	Replace batteries (always both batteries at the same time)
Battery is not charging or Battery indicator shows after Charge empty Battery on	Battery defective	Replace batteries
	Charger error	Check power supply, check all plug connections / contact dealer if the problem cannot be solved
	Fine fuse on charger defective	Replace fuse
	Error in the charging socket	Check whether the charger plug is firmly connected to the charging socket / if necessary. Contact dealer
	Battery is not firmly seated in the electric vehicle	Check the location of the battery, if necessary remove it and plug it back in securely
	No electricity at the socket	Use a different socket; Check fuses in the house
Battery charging current high	Electric vehicle switched on during charging process	Switch off the electric vehicle
	Battery defective	Replace batteries
Electric vehicle doesn't drive	Device not switched on	Switch on the device
	Freewheel lever not engaged	Check the freewheel lever; the engine and gearbox must be firmly connected to each other
	Battery discharged	Check charge level, charge battery
	Battery deeply discharged*	Replace batteries
	Battery cable loose	Check battery cable and ensure secure plug connection
	Charger connected	Remove charger
	Plug or cable disconnected	Check all plug connections/cable connections and plug them in securely if necessary
	Error in the control system**	Contact dealer
Motor runs unevenly or there is a lot of noise from the motor	Check route: Is there an overload?	Pay attention to incline/maximum gradient
	Maximum weight load exceeded?	Checking the payload
	Engine error	Contact dealer
	Error in the control system**	Contact dealer

* Please note the information in Section 5 (Battery and charging unit)

** The control system ensures safe use of the device. NEVER try to fix the problem on your own!

GB

7. SYSTEM DIAGNOSIS

Your mobility scooter is equipped with a self-diagnosis unit. In the event of problems, an error message is given by the "READY" indicator flashing on the display and simultaneous acoustic signals. The number of consecutive signals indicates which of the following errors is involved.

Acoustic signal information

Number of Signals	Meaning	Error/explanation	Solution
1	Battery charge too low	Battery output voltage too low	To charge the battery
2	Battery output voltage too low	Battery charge too low	To charge the battery
3	Battery too high tension	Overload (too much Incline or too high payload)	Reduce speed on inclines / check payload, if necessary. Remove weight / check battery connections.
4	Electrical too high Electricity	Exceeding the max. Voltage for the motor	Switch off and only switch on again after a few minutes / Check motor and cable connections / If problem persists: contact dealer.
5	Freewheel problems	The freewheel lever is not engaged correctly	Check the position of the freewheel lever / engage the freewheel lever securely (please switch off the device first!)
6	Problems with the Control rocker	When turning on the device is located Speed lever not in neutral position	Leave the lever in the neutral position when switching on / the lever may need to be recalibrated: contact your dealer
7	Problems with the Speed regulation	Problems with the control rocker or the speed controller	Check all plug connections / Are the levers in the neutral position when switching on? / If problems persist: contact dealer
8	Problems with the power supply Motors	Problems with the motor or the motor's power supply	Check plug connections / If all connections are correctly seated: Contact your dealer
9	Other cause of error	Possibly problems with the control unit	Check all plug connections / Switch the device off and on again / If problems persist: Contact your dealer
10	Problems with Pushing/Towing	Too high a speed when pushing/ towing	Adjust speed / switch device off and on again

ATTENTION:

- Please report any problems that cannot be remedied on site to your dealer and have them remedied in a specialist workshop.
- Never use force to open covers or loosen plug connections/connect plugs together.
- When transporting, always switch off the device and decouple the motor using the freewheel lever.

8. STORAGE

The electric vehicle should be stored dry, dust-free and at normal room temperatures. Avoid constant, direct sunlight.

Parking conditions:

- Ambient temperature -20°C to +45°C
- Relative humidity ≤85%

GB

9. CLEANING

Clean the upholstery, main frame and plastic parts of the mobility scooter regularly with mild soap and water. Do not use chemicals, solvents, abrasive cleaners or a high-pressure cleaner, otherwise the surface may be damaged. After cleaning, wipe your mobility scooter well dry. But even if it was used in the rain, the mobility scooter should be dried off. Care should be taken to ensure that no cleaning residue remains on the handles or seat, as these could be slippery when the mobility scooter is subsequently used again. Please also avoid oleic acid and products containing oleic acid, as this will cause the electric vehicle to rust and become unstable over time. Store the mobility scooter in a ventilated, dry place.

10. MAINTENANCE

As a user of the electric vehicle, you are always the first to notice possible damage. Therefore, check the condition regularly. This means the service life of the electric vehicle can be extended. If you discover any defects during the inspection, contact your specialist dealer immediately. The electric vehicle may not be put into operation without professional repairs. We recommend that you have your mobility scooter checked by a qualified dealer at least once a year.

1. Items to be serviced:

Batteries, charger, motor, tires, handles, control element, control cable, connectors, clamps, brake cables, wheel bearings, buttons, foot plate, armrests with locking lever, seat, backrest, adjusting lever, screws, nuts, lights and rear-view mirror.

2. Before every trip and after every maintenance check the following:

- Is there any damage or signs of wear and tear?
- Are the settings correct?
- Does the control element work correctly?
- Are the brakes working properly?
- Are the batteries fully charged?
- Are all the lights working?
- Does the electric vehicle drive straight ahead properly?
- Are all removable parts securely attached?
- Is there visible damage (e.g. to the frame, backrest, tires)?
- Are there any material damages such as cracks?
- Is the seat stable or does it wobble a lot?
- Is the tire pressure correct?

3. Check the following once a month:

- Check tire wear and tire pressure every four weeks. Are all the tires running properly? Make sure there is no dirt built up in the tires. The lifespan of the tires depends on use. Check the tread depth of the tires. It should be at least 1 mm. Defective tires that are worn out must be replaced.
- All moving parts must be checked. Do adjustment mechanisms work smoothly and noiselessly? Squeaking noises should be checked by your specialist dealer.
- Tires should be cleaned regularly.
- Check the screws and nuts of the scooter for tightness.
- Are all plug connections, terminals and cables connected correctly and free of corrosion.

ATTENTION:

Seawater, road salt and sand damage the bearings, cables, plug connections and steering components of the electric vehicle. After using the electric vehicle under the influences mentioned, it must be cleaned thoroughly.

11. ELECTROMAGNETIC INTERFERENCE (EMI)

Mobility scooters can be affected by electromagnetic sources (EMI). This disruptive interference from electromagnetic sources such as radio stations, television stations, amateur radio (HAN) transmitters, walkie-talkies, cell phones and alarm systems can cause the mobility scooter to release its brakes, become independent or move in unintended directions. Permanent damage to the tax system is also possible. The intensity of EM energy can be measured in volts per meter (V/m). Every electric vehicle can withstand EMI up to a certain intensity, called its immunity capacity. The Lion has a noise immunity rating of 20V/m, providing useful protection against common sources of radiated EMI.

GB

Note:

1. Do not use portable CB radios or communication devices such as cell phones when starting the mobility scooter.
2. Stay away from devices that emit electromagnetic waves, such as Radio stations, television stations, amateur radio (HAN) stations, etc.
3. If unintentional movement or brake release occurs, turn off the engine immediately as soon as it is safe to do so.
4. If unintentional movement or brake release occurs, turn off the engine immediately as soon as it is safe to do so.

12. GUARANTEE

2 Year Guarantee

Components of the construction frame including:

Seat plate, fork, seat frame and welds

The drive including:

Differential gear and engine, brakes

1 Year Guarantee

The electric vehicle has an unlimited guarantee for 12 months from the date of purchase against damage resulting from production and material defects. Your legal rights remain unaffected by this guarantee.

The battery and battery charger have a 12 month guarantee. Any attempt to open or disassemble these parts will void the corresponding guarantee claims.

Disclaimer of Warrnety

Excluded from the guarantee are wearing parts and parts that need to be replaced due to wear and tear (tires, belts, lamps, upholstery, plastic trim, carbon brushes, fuses and batteries), as well as product damage resulting from misuse or mishap, for which the manufacturer is NOT responsible is. This warranty does not cover labor charges and after-sales services.

Batteries

The batteries have a 12 month manufacturer's warranty. What is not covered is a gradual deterioration in the performance of the batteries due to prolonged discharge, their exposure to cold for a long time, or their wear and tear as a result of heavy use.

Service and Guarantee service

Guarantee service can be provided by your authorized dealer. For information about after-sales service costs, please contact your authorized dealer.

13. ATTENTION!

Disposal

This device does NOT belong in household waste. For proper disposal, please contact the public collection points in your community. For details about the location of such a collection point and any quantitative restrictions per.

Day/month/year as well as any costs incurred for collection, please refer to the information provided by the respective municipality.

Additional equipment

Only use original accessories!

Seek advice from a healthcare professional

If you are unsure how to use the mobility scooter or have questions about its practicality and ergonomics, you should also ask a healthcare professional for advice.

Reporting serious incidents

All serious incidents that occur in connection with the product must be reported to the manufacturer. The importer and the authorized representative must also be informed. The importer will support you with questions relating to serious incidents.

Subject to change

The manufacturer reserves the right to change weights, dimensions or other technical information shown in these instructions for use without prior notice. All numbers, dimensions, etc. are correct as of April 2024.

Stop use

If there are defects or uncertainties, the electric vehicle must not be used. In these cases, please contact your dealer or the importer.

**We accept no liability for consequential damages.
Technical changes and errors reserved!**

Germany	www.rehashop.de
Austria	www.rehashop.at
Switzerland	www.rehashop.ch
Switzerland/Romandy	www.rehashop.ch/fr
France	www.rehashop.fr

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PROTENO AG