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TABLE OF CONTENTS

	Page
1. Essential Advice	13
1.1 Explanation/symbols in the instructions for use	13
2. Instruction manual	14
2.1 Introduction	14
2.2 Intended use	14
2.3 Safety instructions	14
2.4 What you need to do	14
2.5 What you must not do	14
2.6 Suitability of the product	15
3. Product dimensions and specifications	15
3.1 Rollator overview	15
4. Assembly instructions	16
4.1 Operating the brakes	17
4.2 Brake adjustment	17
4.3 Adjusting the handles	18
5. Maintenance	18
6. Maintenance schedule	18
7. Caution	19
8. Safety instructions and warranty	19
9. Attention!	20

Thank you for your trust in our product!

For a safe handling of your new Butterfly Premium, please pay special attention to the following instructions for use and safety precautions. If you have any questions, feel free to contact our customer service team. You can reach our service personnel at the toll-free hotline for your respective country.

Germany:	+49 (0)6421 - 30 30 800
Austria:	+43 (0)1212 - 62 84
Switzerland:	+41 (0)32 - 372 7550
Switzerland/Romandy:	+41 (0)32 - 372 7550
France:	+33 (0)277 - 41 20 96
Netherlands:	+31 (0)30 - 88 93 0 92

We wish you a lot of joy with your new rollator!

Your rehashop team

1. Essential Advice

Before using the rollator for the first time, read these instructions carefully and follow the safety instructions to ensure proper use.

Keep this user manual for future reference and pass it on to other users.

Any liability is excluded in the event of non-observance of the safety rules stated in the instructions for use.

If these instructions for use are not followed and/or the safety measures are not observed, this can lead to accidents and serious injuries.

1.1 Explanation/symbols in the instructions for use

Follow the instructions for use!

Manufacturer

Order number

This appliance should NOT be disposed of with household waste

Lot designation

European authorised representative

Instructions for use contain important safety-related information

The Butterfly Premium is compliant with Regulation (EU) 2017/745

The Butterfly Premium is a class I medical device

Allergy advice: The high-quality rubber handles and wheels contain natural rubber latex. Natural rubber latex can cause allergic reactions up to anaphylactic shock.

Foshan HCT Medical Equipment CO., LTD
No. 11, Dongyang Si Road, Jinsha Danzao,
Nanhai District, Foshan City China
Tel: 0086-757-86611769
Web: www.hct-medical.com

IMPORTEUR

PRO-TENO GmbH

Niederwettertsche Str.1
35094 Lahntal
Germany

Tel: +49(0)6421-30 30 800
Fax: +49(0)6421-30 30 80599
E-Mail: info@reashop.de

2. INSTRUCTION MANUAL

Lightweight Rollator Butterfly PREMIUM

2.1 Introduction

Our rollators are made of high quality materials. They have an integrated seat, bag, parking brakes and height-adjustable push handles. The rollators can be easily folded for transport or storage.

2.2 Intended use

Rollators are intended for locomotion and support the user with limited mobility when walking, sitting and shopping or when stowing small shopping items. The rollator can be used indoors and outdoors on even surfaces. Please pay attention to the maximum load of the walking aid and use the rollator only as a single person, never in pairs or with several people.

The Butterfly Premium must not be used if the user is physically or mentally incapable of using the rollator safely. If there is any doubt about this, please consult your dealer or the person prescribing the product. It is important that the user reads and understands the safety instructions in the user manual before using the product to avoid possible injury to themselves or bystanders, as well as damage to the rollator or surrounding property.

2.3 Safety instructions

The rollators are designed to serve as an aid. They support the user in walking safely. Please read these instructions carefully before using the rollator.

2.4 What you need to do:

1. Make sure that the brakes are always applied when the rollator is left unattended.
2. Make sure that all wheels and castors are in good working order before use.
3. Check that the brakes are in good working order.
4. Check that the height-adjustable handles are secure.
5. Make sure that the rollator is completely unfolded.
6. Please take care when carrying the rollator when it is folded up.
7. Keep your hands away from the folding mechanism to avoid the risk of crushing.

2.5 What you must not do:

When using the Butterfly Premium, there are residual risks that can occur with rollators in general. Some of these are listed below:

1. Do not use the rollator as a wheelchair. A rollator is a walking aid and must not be misused.
2. Do not sit on the seat of the rollator if the brakes are not locked. The rollator could move unintentionally, resulting in injury to the user or bystanders.
3. Do not fall into the seat of the rollator, but make sure you sit down in an orderly manner. If this is not possible under your own power, please ask a third person to help you.
4. Do not load the handbrakes and push handles with various objects. This could affect the functionality of the brakes and/or the stability of the rollator.
5. Undesirable side effects, which can occur with other medical products, are not known with the Butterfly Premium.

2.6 Suitability of the product

The Butterfly Premium can be used by the intended user group, taking into account the contraindications.

3. PRODUCT DIMENSIONS AND SPECIFICATIONS

Total height	Overall length	Total width	Seat depth	Seat width	Seat height	Folded	Size of castors	Weight	Max. Load
									
Min. 78 cm/ Max. 91 cm	68 cm	67,5 cm	25 cm	46 cm	53,5 cm	40 cm	19 cm	8 kg	130 kg


3.1 Rollator overview

4. ASSEMBLY INSTRUCTIONS

The Butterfly Premium is delivered to you folded up.

1. Take the rollator out of the packaging, loosen the screw knobs on the wheel forks of the front castors and place the back strap in its position.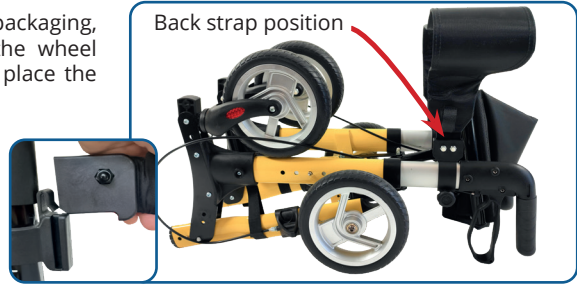

2. Place the Butterfly on the ground with the handles as shown on the right.

3. Fold the front wheel down from the folded position and align the hole in the wheel fork with the holes in the frame.


4. Insert the screw knob and fix the wheel in position. Repeat the steps for the other front wheel.

5. Now place the butterfly on the ground with the handles facing upwards, as shown on the right.

6. Push the locking knob towards the front wheel with your right hand and hold it in position while folding the leg upwards with your right hand. Then release the locking knob so that the bar locks into the lower position. Repeat with the other leg.

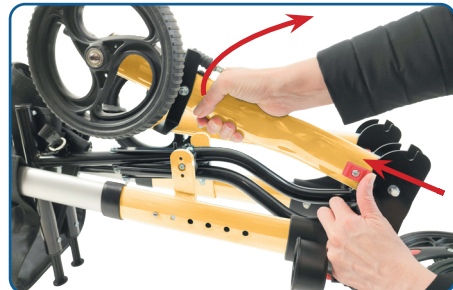- Stand the rollator up and push it apart until it locks into place.

4.1 Operating the brakes

- To brake the rollator while walking, pull both brake levers up towards you.
- To activate the parking brake, push both brake levers down until they engage. The parking brake is released by pulling the brake levers upwards again.


 In any case, the parking brake must be applied before using the seat. Please check that the parking brake has been properly adjusted before sitting down.

4.2 Brake adjustment

Adjust the brake as follows:

Turn the adjustment screw anticlockwise to increase the tension. You fix the adjustment by turning the lock nut clockwise and holding the adjustment screw.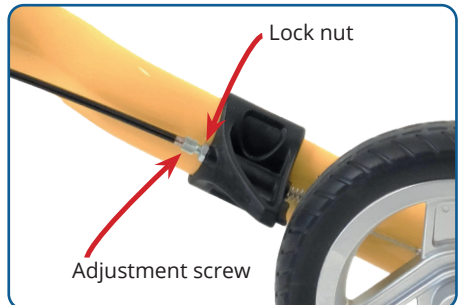

4.3 Adjusting the handles

The handles of the rollators are height-adjustable and can thus be adapted to different needs. Press the button on the outer frame and push or pull the handle up or down to the height that suits you. Let it click into place. Please repeat the process on the other side and make sure that the handles are at the same height and securely engaged.

 The knob must be at the same height on both sides.

5. MAINTENANCE

It is possible that brake pads may extend as a result of wear and tear. If this is the case and appropriate adjustments can no longer be made, it is time to replace the pads. Please contact your specialist dealer for this. Each time before you use the rollator, make sure that the castors are working properly. Also check that the brakes are working properly and that the push handles and buttons are engaged. In the event that any parts on the rollator appear loose or defective, please discontinue use until an authorised dealer has inspected the rollator.

6. MAINTENANCE SCHEDULE:

Weekly

Check the brakes for malfunction. Check the wheels and castors for loose bolts, nuts and cables.

Regularly

Check the handles for signs of wear and if the bolts are tight (on the frame at the front and on the rear axle). Check the wheels for signs of wear.

Annually

Please have your rollator serviced by a specialist dealer once a year. Parts to be serviced: Wheels, handles, brake cable, brake lever, bearings, knobs, bag, screws and nuts.

Cleaning

To clean your rollator, please use warm soapy water and a soft cloth or sponge, which should be dampened accordingly. Household antibacterial cleaner for surfaces can also be used. Care should be taken that no cleaning residues remain on the handles or seat, as these could then be slippery when the rollator is used again. Please do not use scouring pads or scrapers, as these can damage the paintwork. Please also avoid oleic acid and agents containing oleic acid, as this will cause the rollator to rust and become unstable over time. Store the rollator in a ventilated, dry place.

7. CAUTION

1. Maximum load 130 kg.
2. Do not stand on the product.
3. Make sure that both push handles are at the same height.
4. If you have any questions about adjustment or use, please ask your dealer.
5. Please keep a safe distance from pedestrians and other objects with your rollator.
6. Some of the components may cause damage if they come into contact with other objects.
7. please do not replace the wheels and castors of the rollator on your own, but contact your dealer for maintenance. Please contact your dealer for service.
8. Make sure that the rollator is fully unfolded before using it. The scissor arm must be fully unfolded.
9. Make sure that your fingers are not in the area of the scissor arm when folding to avoid injury.

8. SAFETY INSTRUCTIONS AND WARRANTY

- These instructions for use are intended to familiarise you with the operation of this product. Therefore, always keep these instructions in a safe place so that you can refer to them at any time.
- When you purchase this product, you receive a two-year warranty against defects when used properly.
- Please use the product only in its intended manner. Any other use may result in damage to the product or its surroundings.
- Conversion or modification of the product impairs product safety.
Caution: Risk of injury!
- Never carry out repairs yourself.
- Handle the product with care. It can be damaged by knocks, blows or falling from even a small height.
- Keep the product away from moisture and extreme heat.

9. ATTENTION!

Disposal

This appliance does NOT belong in household waste. For proper disposal, please contact the public collection points in your community. For details on the location of such a collection point and on any quantity restrictions per day/month/year as well as on any costs incurred for collection, please refer to the information provided by the respective municipality.

Stop of use

The Butterfly Premium must not be used if it is defective or unsafe. In such cases, please contact your dealer or the importer.

Additional equipment

Only use original accessories!

Seeking advice from a health professional

If you are unsure how to use the Butterfly Premium or have questions about its practicality and ergonomics, you should also seek advice from a health professional.

Reporting of serious incidents

All serious incidents related to the device shall be reported to the manufacturer and the competent authority where the user and/or patient is established. The importer and the authorised representative must also be informed. The importer will assist you with questions related to serious incidents.

MD Conformity of the Butterfly Premium

The Butterfly Premium is a class I medical device and complies with the requirements of the European Medical Device Regulation (EU) 2017/745. A corresponding EU declaration of conformity is available.

No liability is accepted for consequential damage.
Technical changes and errors excepted!

Germany	www.rehashop.de
Austria	www.rehashop.at
Switzerland	www.rehashop.ch
Switzerland/Romandie	www.rehashop.ch/fr
France	www.rehashop.fr
Netherlands	www.rehashop.nl

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PRÒTENO GmbH