PROTENNO

Rollator Vita

Modell: HZ-WA203

DE

GB

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User manual

REF HZ-WA203

Stand 02-2025 – REV001

MD

CE

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TABLE OF CONTENTS

	page
1. IMPORTANT NOTES	30
1.1 Explanation / Symbols in the instructions for use	
1.2 Explanation of the type plate	
1.3 Barrier-free instructions for use	
1.4 Customer service	
1.5 Economic operators	
2. INSTRUCTIONS FOR USE	32
2.1 Declaration of conformity	
2.2 Intended purpose	
2.3 Contraindications	
2.4 Service life	
3. SAFETY INSTRUCTIONS	33
4. PRODUCT DESCRIPTION	36
4.1 Contents	
4.2 Reflectors	
4.3 Product dimensions and specifications	
4.4 Material	
4.5 Rollator overview	
5. INSTALLATION INSTRUCTIONS	39
5.1 Remove packaging	
5.2 Unfolding / folding the rollator	
5.3 Sliding handles »Preparation«	
5.4 Sliding handles »Attachment and adjustment«	
5.5 Luggage basket / shopping basket	
5.6 Storage tray	

	page
6. HANDLING	41
6.1 Seat board	
6.2 Wheels	
6.3 Tipping aid	
6.4 Overcoming obstacles	
7. BRAKING FUNCTION	43
7.1 Service brake	
7.2 Parking brake	
7.3 Brake adjustment	
7.4 Adjusting the braking force	
8. CARE AND HYGIENE	45
8.1 Cleaning	
8.2 Disinfection	
8.3 Hygiene plan	
8.4 Reuse	
9. MAINTENANCE	46
9.1 Maintenance table	
9.2 Parts to be serviced:	
9.3 Check the following before every journey:	
9.4 Check the following once a month:	
10. STORAGE	48
11. WARRANTY	48
12. ATTENTION!	49

GB

1. IMPORTANT NOTES

1.1 Explanation / Symbols in the instructions for use

-  Follow the instructions for use!  Manufacturer
-  Order number  Date of manufacture
-  Batch number  European authorised representative
-  Importer  Swiss authorised representative
-  The instructions for use contain important safety-related information
-  This product is in conformity with Regulation (EU) 2017/745
-  This product is a Class I medical device
-  **Allergy warning:** The high-quality rubber handles and wheels contain natural rubber latex. Natural rubber latex can trigger allergic reactions, including anaphylactic shock.
-  Do not use the product at temperatures below -35 °C or above +60 °C
-  This product does NOT belong in household waste
-  This product is designed for indoor and outdoor use

1.2 Explanation of type plate

- | | |
|--------------------------------------|------------------------------|
| 1. Manufacturer | 6. UDI-DI barcode |
| 2. Model | 7. manufacturer's date |
| 3. Total weight | 8. symbols (see chapter 1.1) |
| 4. Maximum load | 9. SN (serial number) |
| 5. UDI-DI (Unique Device Identifier) | 10. SN barcode |

1.3 Barrier-free instructions for use

 These instructions for use contain all the necessary information for adjusting and operating the rollator. The latest edition of the instructions for use is available online as a PDF at [www.rehashop.de](https://rehashop.de) or can be requested from our customer service department. Larger formats can be provided for visually impaired persons. Blind patients must be instructed by their companions.

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GB

1.4 Customer service

If you have any questions, please contact our online customer service team at info@proteno.de or contact our service staff on the free hotline for the respective country:

- | | |
|-----------------------|-------------------------|
| Germany: | +49 (0)6421 - 30 30 800 |
| Austria: | +43 (0)1212 - 62 84 |
| France: | +33 (0)277 - 41 20 96 |
| Switzerland/Romandie: | +41 (0)32 - 372 7550 |

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2. INSTRUCTIONS FOR USE

2.1 Declaration of conformity

MD The rollator is a class I medical device and fulfils the requirements of the European Medical Device Regulation (EU) 2017/745. A corresponding EU declaration of conformity is available.

2.2 Intended purpose

 The rollator supports people with restricted mobility, e.g. people with walking disabilities or severely restricted walking time, in their mobility and thus allows them a greater radius of action. This rollator is designed for use both indoors and outdoors. It is foldable and therefore handy and space-saving to transport on excursions and when travelling.

2.3 Contraindication

 The rollator should not be used by persons with sensory impairments, severe balance problems or an inability to sit. The user must have sufficient cognitive and physical abilities to be able to operate a rollator.

2.4 Service life

 This product is designed for a service life of 5 years. It is a prerequisite that the product is used as intended and that the instructions for use and maintenance are followed regularly. Careful handling, regular care and correct use can significantly extend the service life. However, improper use and excessive strain can considerably shorten the service life. Please note that this information does not constitute an additional guarantee.

3. SAFETY INSTRUCTIONS

If the safety instructions contained in the operating instructions are not observed, no liability will be accepted. You must therefore the following safety instructions:

SAFETY INSTRUCTIONS

- Talk to your doctor or specialist dealer about whether this rollator is best suited to you and your needs.
- Before each use, check that the rollator is in perfect condition. To do this, check the following components:
 - Are all bolts, nuts and other components tight?
 - Are the wheel surfaces intact
 - Are the height adjustment screws firmly tightened
 - Do the brakes function as parking brakes and service brakes?
 - Do the frame, the weld seams and the plastic elements show no obvious wear or tear
 - Are the wheels and wheel forks free of dirt and can they be turned easily?

A damaged or defective rollator must not be used!

- Before use, make sure that the folding lock under the seat is audibly and tangibly engaged.
- Always activate the parking brake when you stop, park the rollator, sit on it or get up from the rollator. Always apply the service brake slowly, evenly and on both sides.
- Wear on the tyre surface can reduce the braking effect. Check the braking effect of the brakes regularly.
- The rollator may only be used on firm and level ground. There is a risk of slipping on dirty, wet, frosty or snow-covered surfaces! The braking effect is also limited. Adjust your driving and walking behaviour to these dangers.
- On smooth surfaces such as tiles, stone floors, PVC etc., the wheels may lock when braking. You should therefore carefully check the braking behaviour of your rollator on these surfaces and adjust your driving and walking behaviour accordingly. If you want to sit on the rollator, you should also make sure that the wheels have sufficient grip on the surface when the brakes are applied and that the rollator cannot slip.
- The rollator must not be used as a support when standing up from a squatting or sitting position!

GB

SAFETY INSTRUCTIONS

- The maximum permissible load of the rollator is 100 kg. This includes the maximum luggage load of 10 kg.
- The basket included in the scope of delivery is designed for a maximum luggage load of 5 kg. The basket must not be overloaded. Only use the basket for transporting objects.
- The rollator must not be used for purposes other than its intended use or for transporting objects or persons.
- Ensure that the handles are set to the correct height for the user to prevent incorrect loading or falls. Always hold both handles, otherwise you could lose your balance. Walk between the handles. When walking, be careful not to hit the wheels with your feet.
- A maximum of 5 kg may be transported on the tray supplied.
- Take care not to get your hands or clothing into the moving parts of the rollator! Be particularly careful when folding and unfolding the rollator. This applies in particular to the hinges under the seat.
- The walker should be used on the pavement. The rollator must not be used in moving traffic. For your personal safety in road traffic, it is advisable to wear additional reflectors on your clothing in addition to the reflectors already attached to the rollator.
- Do not sit on the rollator when travelling on public transport. Park the rollator with the brake locked so that it cannot fall over or roll away. Find a nearby seat.
- If you have to cross (tram) rails or grilles, drive over them at a right angle and as quickly as possible. Make sure that the wheels do not get caught in the rails or grating.
- Be careful around children. Children must not play with the rollator!
- Do not hang any bags or pouches on the handles as this may affect the balance of the rollator.
- Do not make any adjustments or changes that could impair the safety of the design. Components such as wheels, handles or brake cables may only be replaced by a specialist dealer.
- Only use accessories authorised by the manufacturer. Please also observe the operating instructions for the respective accessories.
- Do not carry or pull the rollator by the brake cables.
- Keep the rollator away from naked flames.
- In strong sunlight, parts of the rollator, e.g. handles or seat, can heat up. In these cases, skin contact should be avoided to prevent burns. Make sure you cover your rollator or park it in a shady place.
- Avoid skin contact with the metal parts of the rollator if it has been exposed to low temperatures for a long time in order to prevent frostbite.

SAFETY INSTRUCTIONS

- Avoid collisions between the rollator and door frames, furniture, etc. Check that the rollator is stable when parked. If the rollator falls over, this can cause damage to the frame, connecting elements and welded seams.
- Avoid loading the folded rollator! This could damage the frame, connecting elements and weld seams.
- Do not use the rollator on (wheelchair) steps.
- Do not stand on the rollator!
- Do not lean on the brake levers!
- If you are unsure or have any questions, please contact your specialist dealer..

ATTENTION: For further important safety instructions, please refer to the relevant chapters in the operating instructions!

4. PRODUCT DESCRIPTION

The rollator is delivered almost fully assembled in a box. Before using the product for the first time, check it for possible transport damage. Pay particular attention to the wheels and wheel suspensions, screw and plastic connections, and brake cables, making sure they are intact. Check the delivery contents for completeness and ensure that they are not damaged. In the event of damage, do not use the product and inform your specialist dealer first. After unpacking, keep the box and packaging material for later storage if possible, or dispose of it properly.

4.1 Contents

The rollator's scope of delivery includes:

- 1 Rollator
- 1 Shopping basket
- 1 Storage tray
- 2 Locking levers
- 2 Screws with washers
- 1 Walking stick holder
- 1 Instruction manual
- 1 Tipping aid

4.2 Reflectors

The rollator is equipped with two reflectors on the rear wheel, front wheel and side frame, which significantly improves the visibility of the user for drivers, cyclists and other road users. These reflectors reflect the light from headlights and other light sources, thus ensuring increased safety in low light conditions.

NOTE: Additional reflectors on your clothing increase visibility at dusk or in the dark, thus reducing the risk of accidents.

4.3 Product dimensions and specification

Model	HZ - WA203
Overall length	665mm
Total width	600mm
Folding size (L x W x H)	ca. 665mm x 790mm x 370mm
Total height/handle height (height adjustable)	790mm - 940mm
Distance between handles	400mm
Seat width	380mm
Seat depth	180mm
Seat height	610mm
Own weight with accessories	10kg
Own weight without accessories	8,9kg
Max. load	100kg/1person
Max. payload Basket/Bag	5kg
Max. load on storage tray	5kg
Tire diameter	200mm
Tropic	1150mm diameter Ø
Frame color	blue
Recommended height	1600mm - 1950mm

4.4 Material

Frame	Steel
Painting	Powder coating
Handles	Chrome plated
Handles	PVC (polyvinyl chloride)
Seat	Wood + PU (polyurethane)
tray	PET (polyethylene terephthalate)
Basket	Steel + PVC (polyvinyl chloride)
Tires	PP (Polypropylene) + TPR (Thermoplastic Elastomer)

***Note:** All metals used are corrosion resistant.

4.5 Rollator Overviews

- | | |
|---|--|
| 1. Push handle  | 10. Seat board |
| 2. Brake | 11. Adjustment screw (brake) |
| 3. Parking lever (handle height) | 12. Luggage basket  |
| 4. Folding mechanism (folding lock) | 13. Walking stick holder |
| 5. Cross strut | 14. Luggage rack |
| 6. Rear wheel rigid  | 15. Front castor  |
| 7. Brake cable  | 16. Tipping aid |
| 8. Storage tray | 17. Reflectors |
| 9. Cane clamp | |

16. Tipping aid

9. Cane clamp

5. INSTALLATION INSTRUCTIONS

5.1 Remove packaging

The rollator will be delivered to you already folded. Take the rollator out of the packaging and remove all packaging material and the protective film on the 4 wheels.

 **ATTENTION:** Make sure that no clothing or body parts get caught when folding and unfolding.

1. Pull the rear wheels back

5.2 Unfolding/folding the rollator

UNFOLDING

Pull the rear wheels back. Then push the crossbar straight down. Next, fold down the folding lock until it clicks into place.

FOLDING

To fold the rollator, release the folding lock, pull up the crossbar and push the rear wheels against the frame.

 **ATTENTION:** Check that the folding lock is securely in place before use to prevent the rollator from folding up unintentionally.

 **ATTENTION:** Make sure that no clothing or body parts get caught when folding and unfolding.

2. Push the cross strut downwards

3. Folding lock engaged

5.3 Sliding handles »Preparation«

Insert the screw on the inside of the guide frame and place the washer as shown in the illustration. Now place the locking lever at the end of the screw and turn it clockwise a little.

 **IMPORTANT:** Always attach the locking lever so that it sits on the outside of the guide frame.

5.4 Sliding handles

»attachment and adjustment«

Insert the two handles on the right and left at least 10 cm into the guide frame and secure them with the two locking levers. Adjust the height of the push handles to suit your height.

TO TIGHTEN: turn clockwise

TO LOOSEN: turn anti-clockwise

 **IMPORTANT:** Then align the locking levers so that their ends point downwards. To do this, pull the locking levers outwards and turn them as shown. This does not change the locking effect of the levers!

5.5 Luggage basket / shopping basket

The basket offers a comfortable way to transport your items or purchases. It can be hooked into the hooks on the left and right below the seat board.

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5.6 Storage tray

The tray is inserted with the feet into the »holes« of the seat board. It is only intended for the storage of small items. Do not place any hot, sharp or heavy objects on it (max. load 5 kg).

 **IMPORTANT:** Make sure that the »feet« of the storage tray are inserted evenly into the holes in the seat board.

 **ATTENTION:** The tray must not be used as a seat!

6. HANDLING

6.1 Seat board

The seat allows you to take short breaks to rest your legs when you are in the correct sitting position. Do not lean sideways or backwards over the rollator when sitting. This could affect the stability of the rollator.

 **ATTENTION:** Make sure that the brakes are always applied on the right and left before you sit down (see chapter 7.2 Parking brake).

 **ATTENTION:** The board is not a substitute for a chair! Only place weight on it vertically and do not lean back or against it at an angle!

 **ATTENTION:** Only sit on the seat with your back to the basket. Do not move the rollator when someone is sitting on the seat board. The seat has been tested for a user weight of up to 100 kg.

GB

6.2 Wheels

The rollator is supplied with puncture-proof solid rubber wheels. It is therefore not necessary to check the air pressure. If the wheels are worn or too old, they should be replaced by your specialist dealer. Make sure that the wheels can turn freely and are not damaged.

6.3 Tipping aid

The Vita has a tipping aid that makes it easier to tilt the rollator.

 **ATTENTION:** Before you overcome obstacles, make sure that you can stand securely on one leg. If necessary, practise the application with a helper before you do it independently.

6.4 Overcoming obstacles

1 To tilt the rollator for flat steps and kerbs, approach the obstacle with the rollator straight on until the two front wheels are parallel directly in front of the obstacle. Pull the brake levers 2 up with both hands so that both rear wheels are braked.

Place your foot slowly and in a controlled manner on the tipping aid 3 and at the same time pull both handles 4 towards you, to bring the rollator safely into the tipping position.

Once the rollator is balanced on the rear wheels, release the right and left brake levers lightly in turn and apply them again to move the rollator forward step by step on the rear wheels.

When both rear wheels are parallel and close to the obstacle, apply both brake levers evenly. Now you can lower the front wheels onto the obstacle in a controlled manner while the brakes remain applied.

Carefully release the brakes and slowly push the rollator forward until it is standing securely on the obstacle or the obstacle has been completely overcome.

7. BRAKING FUNCTION

7.1 Service brake

The service brake is used to slow down while riding. The speed is reduced by pulling up on the brake lever. The service brake should always be applied evenly and slowly on both sides to avoid falls.

! **IMPORTANT:** The rollator should not be pushed for long periods with the parking brake applied. This can cause the tyres and brakes to wear particularly quickly and impair their function.

! **ATTENTION:** Check the function of the brakes regularly. We recommend a check before each journey. The rollator must not be used with damaged brakes or damaged wheels.

GB

7.2 Parking brake

! The parking brake is activated by pressing the brake lever down until it engages. When applied correctly, the brake ensures that the rollator cannot roll away. Both parking brakes must always be applied and engaged. To release the parking brake, briefly pull the brake lever up again.

! **ATTENTION:** The parking brakes are not designed as service brakes! Therefore, the parking brake must not be applied while driving. Tyres and brakes can wear particularly heavily and their function can be impaired.

! **ATTENTION:** The parking brakes must always be applied on both sides and properly engaged before you use the rollator for support or the seat board for relief.

! **ATTENTION:** The effectiveness of the brakes can be affected by wear to the solid rubber rollers. The braking effect should therefore be checked at regular intervals.

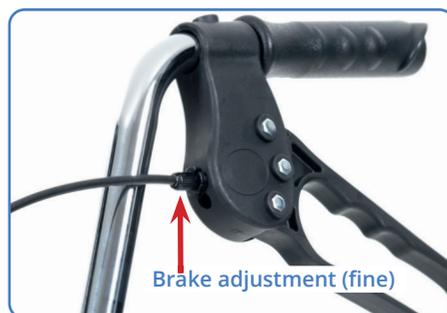
7.3 Brake adjustment

⚠ If the braking force decreases due to wear on the wheels, the brake should be readjusted. We recommend having the brake settings carried out by your specialist dealer.

The brake must be adjusted so that both wheels brake evenly. The distance between the brake bolt and the wheel surface should be about **8 mm**.

Adjust the rear wheels as described in chapter 7.4. Use the cable tensioner and the locknut on the brake levers to fine-tune the brake.

⚠ **ATTENTION:** After replacing the wheels, the brake adjustment must also be checked and readjusted if necessary.

7.4 Adjusting the braking force

1. Loosen the locknut.
2. Adjust the brake force by turning the cable tensioner inwards or outwards.
3. Check the brake function by turning the wheels and operating the brake levers. The wheels should turn freely, but stop immediately when you apply the brakes. When the brake is applied, the wheels must not move.
4. Retighten the locknut firmly.

⚠ **ATTENTION:** If you are unsure, please contact your specialist dealer for help in setting the brake correctly.

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8.1 Cleaning

- Clean the rollator regularly and after heavy soiling with lukewarm water and neutral, commercially and domestically available cleaning agents. After using cleaning agents, the rollator must be thoroughly rinsed with clean water. Wipe your rollator dry with a clean cloth after every cleaning, even if the rollator was used in the rain.
- Also clean the brake bolts and springs regularly. Care should be taken to ensure that no cleaning residue remains on the handles, brake bolts or similar, as these may become slippery or the braking effect may be impaired.
- Never use abrasives or a high-pressure cleaner, as this may damage the surface. Please also avoid oleic acid and agents containing oleic acid, as these will cause the rollator to rust and become unstable over time.
- Store the rollator in a dry, well-ventilated place.
- Do not use the rollator at temperatures below -35°C or above $+60^{\circ}\text{C}$.

⚠ **ATTENTION:** Seawater, road salt and sand can damage the bearings and steering components of the rollator. If the rollator is used in such conditions, it must be thoroughly cleaned.

⚠ **ATTENTION:** When cleaning, water must not get into the tubes of the rollator. Therefore, the rollator must only be cleaned when fully assembled.

8.2 Disinfection

1. Put on protective gloves and, if necessary, protective clothing (disposable apron, face mask, etc.) before preparing the rollator.
2. Always clean the device first (see chapter 8.1).

⚠ **ATTENTION:** Before disinfection, there must be no cleaning agent residue on the surface.

3. Wipe down the rollator with a DGHM-/VAH-listed disinfectant or spray it on (observe the exposure time according to the manufacturer's instructions). Wipe disinfection is generally preferable to spray disinfection. After the disinfectant has taken effect, dry the rollator with a clean, dry cloth.

⚠ **ATTENTION:** Do not use disinfectants containing chlorine or phenol, or vinegar-based cleaning agents.

GB

8.3 Hygiene plan before each patient change

Before each re-use or change of patient, the rollator must be prepared as follows:

1. **Cleaning** (Follow the instructions in Chapter 8.1)
2. **Disinfection** (Follow the instructions in chapter 8.2)

 **ATTENTION** The disinfection of the rollator during a change of patient may only be carried out by a specialist.

8.4 Reuse

The product is suitable for reuse if it has been cleaned and disinfected in accordance with the instructions in chapter 8.3. In addition, it must be inspected by a specialist dealer for damage and then released for reuse. Have your rollator serviced by your specialist dealer at regular intervals. They will detect any wear and replace components if necessary. If you notice any faults during the regular inspection of your rollator's braking force, please contact your specialist dealer immediately and in confidence!

9.1 Maintenance table

What needs to be checked?	Check for function	 Notes
General condition	Check all frame parts, including the welds, for deformation, damage, wear and corrosion.	Defective product parts must be replaced before reuse.
Fold-protection	Check that the folding lock engages properly (see chapter 5.2)	If the folding mechanism is not working properly, it must be repaired or replaced.
Fasteners on tight fit	Check, for example, whether the locking lever with the screw is properly fixing the push-handle. (See chapter 5.4)	If any loosened or missing screws, nuts, rivets, etc. are found, they must be tightened, added or replaced.
Functional test of the brake system	Apply the service brake as far as it will go. The wheels should now no longer turn when pushed forcefully. Apply the parking brake by pressing the brake lever down until it engages. The wheels should now no longer turn when pushed forcefully.	If a brake is faulty, the brake play, brake cable, brake shoes and wheels must be checked and, if necessary, the defective part replaced.
Roller profile and ball bearing	Check the rollers for heavy wear/defects and check the ball bearing for damage.	If there is wear, defects or cracks, the rollers or ball bearing must be replaced.
Wheels (PU) Function	The castors must roll straight, easily and quietly and must not wobble. Check that the wheelchair rolls straight ahead.	If damaged or heavily worn, the rollers must be replaced.
Push handles	The handgrips must be firmly attached, the push handles must be at the same height and must not show any signs of damage or bending. The locking screws must lock securely.	Serious defects to handles/push handles or locking screws must be rectified.
Cleaning	Check for dirt	The entire wheelchair should be cleaned depending on how dirty it is, but at least once a month (see chapter 8. Care and hygiene).

GB

9. MAINTENANCE

As a user of the rollator, you are always the first to notice any damage. Check the condition of the rollator regularly. If you notice any defects during your inspection, contact your specialist dealer immediately. The rollator must not be used without a professional repair. We recommend that you have your rollator checked by a qualified dealer at least once a year.

9.2 Parts to be serviced:

Frame parts, wheels, brake cables, brake lever, brake bolts, push handles, seat, folding lock, storage tray, luggage basket, locking lever, bearings, knobs, screws and nuts.

9.3 Check the following before every journey:

- Do the brakes work properly?
- Do the castors work properly? Test whether the rollator rolls straight ahead.
- Are all removable parts securely attached (basket, tray)?
- Is there any visible damage (e.g. to the frame, wheels, brake cables, brake levers)?
- Is there any material damage such as cracks?

9.4 Check the following once a month:

- Check the tyre wear every four weeks. Do all wheels run smoothly when idling? Make sure that no dirt has accumulated in the wheel hubs. The service life of the wheels depends on use. Defective wheels with wear must be replaced.
- All moving parts must be checked. Does the folding mechanism work smoothly and silently? Squeaking noises should be checked by a specialist dealer.
- Check that the screws and nuts on the rollator are tight.

10. STORAGE

Store the rollator in a dry place, protected from light and dust. Do not store near heat sources. Protect the rollator from dirt or contamination after preparation or if it is not used for a longer period of time.

11. WARRANTY

We provide a warranty for this product in accordance with European statutory provisions. The first 12 months are the warranty period. This warranty covers manufacturing and assembly defects. We are not liable for wear and tear or careless and improper use. The warranty is void if the product is modified or repaired by unauthorised dealers or persons. It does not include costs for troubleshooting, freight, loss of use or earnings, for necessary travel and similar costs. Also not covered by the warranty are damages or injuries resulting from improper use, misuse or failure to follow the operating instructions. It is not permitted to make technical changes to the product.

12. ATTENTION!

Disposal

This device must NOT be disposed of with household waste. For proper disposal, please contact the public collection points in your community. For details on the location of such a collection point and on any daily/monthly/annual quantity restrictions that may apply, as well as on any costs that may be incurred for collection, please refer to the information provided by the respective municipality.

Use stop

In case of defects or uncertainties, the rollator must not be used. In such cases, please contact your dealer or the importer.

Additional equipment

Use only original accessories!

Seek advice from a health professional

If you are unsure about how to use the rollator or have questions about its suitability and ergonomics, you should also consult a healthcare professional for advice.

Reporting of serious incidents

Should a serious incident occur in connection with this medical device, please contact the manufacturer of this product, the importer and the authorised representative.

GB

No liability is accepted for consequential damage.
Subject to technical modifications and errors!

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Proteno GmbH