Leichtgewichtrollator Vigeo

Modell: HZ-WA204

DE

GB

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User manual

REF HZ-WA204

Stand 12-2025 – REV003

MD

CE

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Thank you for your trust in our product!

To ensure safe use of your new rollator, please pay particular attention to the following instructions for use and safety. If you have any questions, please feel free to contact our customer service team (see Chapter 1.4).

We wish you much joy with your new rollator!

Please read these instructions carefully before use, follow the instructions and keep the instructions for use carefully for future reference! These instructions for use are a translation from German. If this aid is used by other people, they must be fully informed of the instructions given here before use. Any liability is excluded if the safety rules stated in the instructions for use are not observed. Failure to follow these instructions for use and/or comply with the safety measures may result in falls and serious injuries.

Video tutorial

To assist you with the assembly, you can find a video tutorial here: scan the QR code.

ATTENTION: This video tutorial does not release you from your obligation to read the instructions for use and follow the safety instructions!

Notes

GB

TABLE OF CONTENTS

	page
1. IMPORTANT NOTES	30
1.1 Explanation / Symbols in the instructions for use	
1.2 Explanation of the type plate	
1.3 Barrier-free instructions for use	
1.4 Customer service	
1.5 Economic operators	
2. INSTRUCTIONS FOR USE	32
2.1 Declaration of conformity	
2.2 Intended purpose	
2.3 Contraindications	
2.4 Service life	
3. SAFETY INSTRUCTIONS	33
4. PRODUCT DESCRIPTION	36
4.1 Contents	
4.2 Reflectors	
4.3 Product dimensions and specifications	
4.4 Material	
4.5 Rollator overview	
5. INSTALLATION INSTRUCTIONS	39
5.1 Remove packaging	
5.2 Unfolding / folding the rollator	
5.3 Sliding handles »Preparation«	
5.4 Sliding handles »Attachment and adjustment«	
5.5 Backrest	
5.6 Luggage basket / shopping basket	

	page
6. HANDLING	41
6.1 Seat board	
6.2 Wheels	
6.3 Tipping aid	
6.4 Overcoming obstacles	
7. BRAKING FUNCTION	43
7.1 Service brake	
7.2 Parking brake	
7.3 Brake adjustment	
7.4 Adjusting the braking force	
8. CARE AND HYGIENE	45
8.1 Cleaning	
8.2 Disinfection	
8.3 Hygiene plan	
8.4 Reuse	
9. MAINTENANCE	46
9.1 Maintenance table	
9.2 Parts to be serviced:	
9.3 Check the following before every journey:	
9.4 Check the following once a month:	
10. STORAGE	48
11. WARRANTY	48
12. ATTENTION!	49


1. IMPORTANT NOTES

1.1 Explanation / Symbols in the instructions for use

-  Follow the instructions for use!  Manufacturer
-  Order number  Date of manufacture
-  Batch number  European authorised representative
-  Importer  Swiss authorised representative
-  The instructions for use contain important safety-related information
-  This product is in conformity with Regulation (EU) 2017/745
-  This product is a Class I medical device
-  **Allergy warning:** The high-quality rubber handles and wheels contain natural rubber latex. Natural rubber latex can trigger allergic reactions, including anaphylactic shock.
-  Do not use the product at temperatures below -35 °C or above +60 °C
-  This product does NOT belong in household waste
-  This product is designed for indoor and outdoor use

1.2 Explanation of the type plate

- | | |
|---|--|
| <ol style="list-style-type: none"> 1. Manufacturer 2. Model 3. Maximum load 4. Total weight 5. UDI-DI (Unique Device Identifier) | <ol style="list-style-type: none"> 6. UDI-DI barcode 7. Manufacturer's date 8. Symbols (see chapter 1.1) 9. SN (serial number) 10. SN barcode |
|---|--|

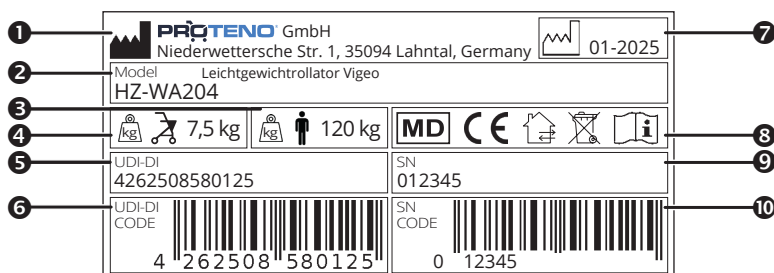1.3 Barrier-free instructions for use

These instructions for use contain all the necessary information for adjusting and operating the rollator. The latest edition of the instructions for use is available online as a PDF at [www.rehashop.de](https://rehashop.de) or can be requested from our customer service department. Larger formats can be provided for visually impaired persons. Blind patients must be instructed by their companions.

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GB

1.4 Customer service

If you have any questions, please contact our online customer service team at info@proteno.de or contact our service staff on the free hotline for the respective country:

Germany:	+49 (0)6421 - 30 30 800
Austria:	+43 (0)1212 - 62 84
France:	+33 (0)277 - 41 20 96
Switzerland/Romandie:	+41 (0)32 - 372 7550

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2. INSTRUCTIONS FOR USE

2.1 Declaration of conformity

MD The rollator is a class I medical device and fulfils the requirements of the European Medical Device Regulation (EU) 2017/745. A corresponding EU declaration of conformity is available.

2.2 Intended purpose

The rollator supports people with restricted mobility, e.g. people with walking disabilities or severely restricted walking time, in their mobility and thus allows them a greater radius of action. This rollator is designed for use both indoors and outdoors. It is foldable and therefore handy and space-saving to transport on excursions and when travelling.

2.3 Contraindication

The rollator should not be used by persons with sensory impairments, severe balance problems or an inability to sit. The user must have sufficient cognitive and physical abilities to be able to operate a rollator.

2.4 Service life

This product is designed for a service life of 5 years. It is a prerequisite that the product is used as intended and that the instructions for use and maintenance are followed regularly. Careful handling, regular care and correct use can significantly extend the service life. However, improper use and excessive strain can considerably shorten the service life. Please note that this information does not constitute an additional guarantee.

3. SAFETY INSTRUCTIONS

If the safety instructions contained in the operating instructions are not observed, no liability will be accepted. You must therefore the following safety instructions:

GB

SAFETY INSTRUCTIONS

- Please consult your doctor or specialist dealer to find out whether this rollator is the best option for you and your needs.
- Before each use, check that the rollator is in perfect condition. To do this, check the following components:
 - Are all screws, nuts and other components tightened
 - Are the wheel surfaces intact?
 - Are the height adjustment screws tight?
 - Do the brakes work as parking brakes and as service brakes?
 - Do the frame, the welds and the plastic elements show no obvious wear?
 - Are the wheels and wheel forks free of dirt and can they be turned easily?

A damaged or defective rollator must not be used!

- Before use, make sure that the folding lock under the seat has audibly and perceptibly engaged.
- Always activate the parking brake when you stop, put the rollator down, sit down on it or stand up from it. Always apply the service brake slowly and evenly on both sides.
- The braking effect can be reduced by wear to the tyre surface. Check the braking effect of the brakes regularly.
- The rollator should only be used on firm, level ground. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also reduced. Adjust your driving and walking behaviour to these dangers.
- On smooth surfaces, such as tiles, stone floors, PVC, etc., the wheels may lock under certain circumstances when braking. Therefore, carefully check the braking behaviour of your rollator on these surfaces and adjust your driving and walking behaviour accordingly.
- If you want to sit on the rollator, you should also make sure that the wheels have sufficient grip on the ground with the brakes applied and that the rollator cannot slip away.
- The maximum permissible load of the rollator is 120 kg. This includes the maximum luggage load of 7 kg.

SAFETY INSTRUCTIONS

- The rollator must not be used as a support when standing up from a squatting or sitting position!
- Be careful not to get your hands or clothes caught in the moving parts of the rollator! Take particular care when unfolding and folding the rollator. This applies especially to the hinges under the seat.
- The walker should be used on the pavement. The walker must not be used in moving traffic. For your personal safety in road traffic, it is recommended that you wear additional reflectors on your clothing in addition to the reflectors already attached to the walker.
- Do not sit on your rollator when travelling on public transport. Set the brake on your rollator and place it so that it cannot fall over or roll away. Find a nearby seat.
- When you have to cross (tram) tracks or a grid, cross them as quickly as possible at a right angle. Make sure that the wheels do not get stuck in the tracks or grid.
- Be careful around children. Children must not play with the rollator!
- Do not hang bags or other items on the handles, as this can affect the balance of the rollator.
- Do not make any adjustments or changes that could compromise the design-related safety. The replacement of components, such as wheels, handles or brake cables, may only be carried out by the specialist dealer.
- Use only accessories approved by the manufacturer. Please also refer to the instructions for use for the respective accessory.
- Do not carry or pull the rollator by the brake cables.
- Keep the rollator away from open flames.
- In strong sunlight, parts of the rollator, such as the handles or seat, can become hot. In these cases, skin contact should be avoided to prevent burns. Make sure to cover your rollator or to store it in a shady place.
- Avoid skin contact with the metal parts of the rollator if it has been exposed to low temperatures for a long time to prevent possible frostbite.
- Avoid crashing the rollator into door frames, furniture or similar obstacles. Check that the rollator is standing securely in its parked position. If the rollator falls over, the frame, connecting elements and welds may be damaged.
- Avoid placing any loads on the folded rollator! The frame, connecting elements and welds could be damaged as a result.
- Do not use the rollator on (rolling) stairs.
- Do not stand on the rollator!
- Do not lean on the brake levers!
- If you are unsure or have any questions, please contact your specialist dealer.

SAFETY INSTRUCTIONS

- The basket included in the scope of delivery is designed for a maximum load of 7 kg. Do not overload the basket. Only use the basket to transport items.
- The rollator must not be used outside of its intended use or to transport objects or people.
- To avoid falls and improper loading, make sure that the handles are adjusted to the correct height for the user. Always hold both handles, otherwise you could lose your balance. Walk between the handles. When walking, be careful not to bump your feet against the wheels.

GB

ATTENTION: For further important safety instructions, please refer to the relevant chapters in the operating instructions!

4. PRODUCT DESCRIPTION

The rollator is delivered almost fully assembled in a box. Before using the product for the first time, check it for possible transport damage. Pay particular attention to the wheels and wheel suspensions, screw and plastic connections, and brake cables, making sure they are intact. Check the delivery contents for completeness and to make sure nothing is missing. If there is any damage, do not use the product and inform your specialist dealer first. After unpacking, keep the box and packaging material for later storage if possible, or dispose of it properly.

4.1 Contents

The rollator's scope of delivery includes:

- 1 Rollator
- 1 Luggage basket
- 2 Screw-on buttons
- 2 Washers/nuts
- 1 Backrest
- 1 Walking stick holder
- 1 User manual
- 1 Tipping aid

4.2 Reflectors

The rollator is equipped with two reflectors on the rear wheel, front wheel and side frame, which significantly improves the visibility of the user for drivers, cyclists and other road users. These reflectors reflect the light from headlights and other light sources, thus ensuring increased safety in low light conditions.

NOTE: Additional reflectors on your clothing increase visibility at dusk or in the dark, thus reducing the risk of accidents.

4.3 Product dimensions and specification

Model	HZ-WA204
Overall length	720 mm
Total width	590 mm
Folding size (L x W x H)	ca. 480 x 590 x 960 mm
Total height/handle height (height adjustable)	825 - 955 mm
Distance between handles	390 mm
Seat width	350 mm
Seat depth	340 mm
Seat height	555 mm
Own weight with accessories	7,5 kg
Own weight without accessories	6,7 kg
Max. load	120 kg/1person
Max. payload basket/bag	7 kg
Max. load on storage tray	7 kg
Tire diameter	190 x 32 mm
Tropic	850 mm diameter Ø
Frame color	grey
Recommended height	1600mm - 1950mm

GB

4.4 Material

Frame	Aluminium
Painting	Powder coating
Handles	Aluminium
Handles	TPR (thermoplastic elastomer)
Seat	Wood + PVC (polyvinyl chloride)
Basket	Steel + PVC (polyvinyl chloride)
Tires	PP (Polypropylene) + PVC (Polyvinyl chloride)

***NOTE:** All metals used are corrosion resistant.

4.5 Rollator Overviews

1. Push handle 
2. Brake
3. Screw knobs (for handle height)
4. Cane clamp
5. Reflectors
6. Folding mechanism (folding lock)
7. Cane holder
8. Adjusting screw (brake)
9. Fixed rear wheel 
10. Tipping aid
11. Brake cable
12. Backrest 
13. Spring button (backrest disassembly)
14. Seat board (upholstered) 
15. Luggage basket
16. Pull rope for folding mechanism
17. Wheel fork with castor screw
18. Front wheel steerable 

5. cane clamp

10. tipping aid

5. INSTALLATION INSTRUCTIONS

5.1 Remove packaging

The rollator will be delivered to you folded. Take the rollator out of the packaging and remove all packaging material and the protective film on the 4 wheels.

GB

5.2 Unfolding / Folding the rollator

UNFOLDING

Place the rollator in front of you so that it is facing in the direction of travel and apply the parking brake (see chapter 7.2). Fold up the seat and push the crossbar straight down until you hear the folding lock engage.

FOLDING

Position the rollator so that it is facing you in the direction of travel. Apply the parking brake (see chapter 7.2) and remove the shopping basket. Fold up the seat and pull up the pull rope. The rollator will fold up.

ATTENTION: Check that the folding lock is securely in place before use to prevent the rollator from folding up unintentionally.

ATTENTION: Make sure that no clothing or body parts get caught when folding and unfolding.


5.3 Sliding handles

»Preparation«

Insert the bolt heads on the outside of the guide frame and place the two washers as shown in the illustration. Now place the nut on the bolt end and tighten it slightly in a clockwise direction.

IMPORTANT: Always attach the locking lever so that it sits on the outside of the guide frame.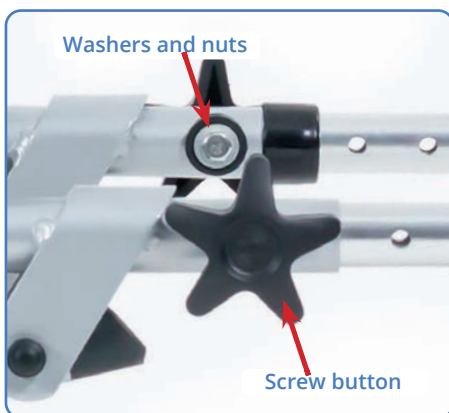

5.4 Sliding handles

»Attachment and adjustment«

Insert the two handles on the right and left into the guide frame by at least 15 cm and secure them with the two screw buttons. Adjust the height of the push handles to suit your height.

TO TIGHTEN: Turn clockwise

TO LOOSEN: Turn anti-clockwise

5.5 Backrest

FIXING

The padded backrest is inserted into the frame using the handles. To do this, press and hold the side buttons. As soon as you hear the pins click into place, you know that the backrest is securely and correctly attached.

REMOVE

Press the buttons on the sides and lift the backrest up and out.

5.6 Luggage basket / Shopping basket

The basket offers a convenient way to transport your belongings or purchases. It is hooked into the hooks on the left and right below the seat board.

ATTENTION: Only use the basket to transport items and do not overload it (max. load 5 kg)!

6. HANDLING

6.1 Seat board (padded)

The padded seat is larger and more comfortable than on most rollators, making it pleasant to sit on even during longer breaks, while also taking the strain off your legs.

 **ATTENTION:** Make sure that the brakes are always applied on the right and left before you sit down (see chapter 7.2 Parking brake).

 **ATTENTION:** Only sit on the seat with your back to the basket. Do not move the rollator when someone is sitting on the seat board. The seat has been tested for a user weight of up to 120 kg.

GB

6.2 Wheels

The rollator is supplied with puncture-proof solid rubber wheels. It is therefore not necessary to check the air pressure. If the wheels are worn or too old, they should be replaced by your specialist dealer. Make sure that the wheels can turn freely and are not damaged.

6.3 Tipping aid

The Vigeo has a tipping aid that makes it easier to tilt the rollator.

 **ATTENTION:** Before you overcome obstacles, make sure that you can stand securely on one leg. If necessary, practise the application with a helper before you do it independently.

6.4 Overcoming obstacles

① To tilt the rollator for flat steps and kerbs, approach the obstacle with the rollator straight on until the two front wheels are parallel directly in front of the obstacle. Pull the brake levers ② up with both hands so that both rear wheels are braked.

Place your foot slowly and in a controlled manner on the tipping aid ③ and at the same time pull both handles ④ towards you, to bring the rollator safely into the tipping position.

Once the rollator is balanced on the rear wheels, release the right and left brake levers lightly in turn and apply them again to move the rollator forward step by step on the rear wheels.

When both rear wheels are parallel and close to the obstacle, apply both brake levers evenly. Now you can lower the front wheels onto the obstacle in a controlled manner while the brakes remain applied.

Carefully release the brakes and slowly push the rollator forward until it is standing securely on the obstacle or the obstacle has been completely overcome.

7. BRAKING FUNCTION

7.1 Service brake

The service brake is used to slow down while riding. The speed is reduced by pulling up on the brake lever. The service brake should always be applied evenly and slowly on both sides to avoid falls.

 **IMPORTANT:** The rollator should not be pushed for long periods with the parking brake applied. This can cause the tyres and brakes to wear particularly quickly and impair their function.

 **ATTENTION:** Check the function of the brakes regularly. We recommend a check before each journey. The rollator must not be used with damaged brakes or damaged wheels.

GB

7.2 Parking brake

 The parking brake is activated by pressing the brake lever down until it engages. When applied correctly, the brake ensures that the rollator cannot roll away. Both parking brakes must always be applied and engaged. To release the parking brake, briefly pull the brake lever up again.

 **ATTENTION:** The parking brakes are not designed as service brakes! Therefore, the parking brake must not be applied while driving. Tyres and brakes can wear particularly heavily and their function can be impaired.

 **ATTENTION:** The parking brakes must always be applied on both sides and properly engaged before you use the rollator for support or the seat board for relief.

 **ATTENTION:** The effectiveness of the brakes can be affected by wear to the solid rubber rollers. The braking effect should therefore be checked at regular intervals.

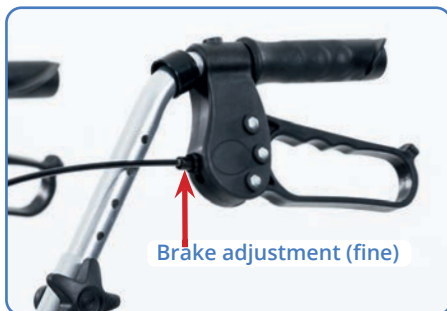
7.3 Brake adjustment

 If the braking force decreases due to wear on the wheels, the brake should be readjusted. We recommend having the brake settings carried out by your specialist dealer.

The brake must be adjusted so that both wheels brake evenly. The distance between the brake bolt and the wheel surface should be about **5 mm**.

Adjust the rear wheels as described in chapter 7.4. To fine-tune the brake, use the cable tensioner and the locknut on the brake levers, as described in chapter 7.4.



 **ATTENTION:** After replacing the wheels, the brake adjustment must also be checked and readjusted if necessary.

7.4 Adjusting the braking force

1. Loosen the locknut.
2. Adjust the brake force by turning the cable tensioner inwards or outwards.
3. Check the brake function by turning the wheels and operating the brake levers. The wheels should turn freely, but stop immediately when you apply the brakes. When the brake is applied, the wheels must not move.
4. Retighten the locknut firmly.

 **ATTENTION:** If you are unsure, please contact your specialist dealer for help in setting the brake correctly.

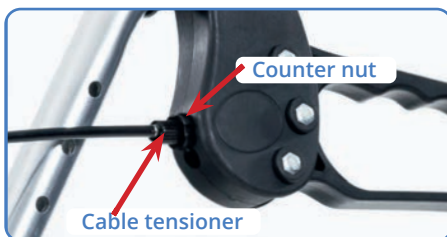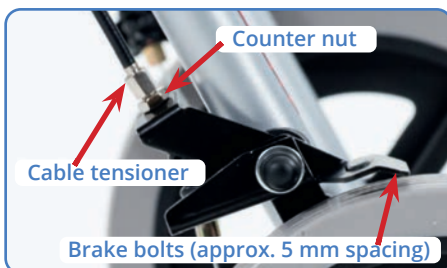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8.1 Cleaning

- Clean the rollator regularly and after heavy soiling with lukewarm water and neutral, commercially and domestically available cleaning agents. After using cleaning agents, the rollator must be thoroughly rinsed with clean water. Wipe your rollator dry with a clean cloth after every cleaning, even if the rollator was used in the rain.
- Also clean the brake bolts and springs regularly. Care should be taken to ensure that no cleaning residue remains on the handles, brake bolts or similar, as these may become slippery or the braking effect may be impaired.
- Never use abrasives or a high-pressure cleaner, as this may damage the surface. Please also avoid oleic acid and agents containing oleic acid, as these will cause the rollator to rust and become unstable over time.
- Store the rollator in a dry, well-ventilated place.
- Do not use the rollator at temperatures below -35°C or above $+60^{\circ}\text{C}$.

GB

 **ATTENTION:** Seawater, road salt and sand can damage the bearings and steering components of the rollator. If the rollator is used in such conditions, it must be thoroughly cleaned.

 **ATTENTION:** When cleaning, make sure that no water enters the tubes of the rollator. Therefore, the rollator should only be cleaned when fully assembled.

8.2 Disinfection

1. Put on protective gloves and, if necessary, protective clothing (disposable apron, face mask, etc.) before preparing the rollator.
2. Always clean the device first (see chapter 8.1).

 **ATTENTION:** Before disinfection, there must be no cleaning agent residue on the surface.

3. Wipe down the rollator with a DGHM-/VAH-listed disinfectant or spray it on (observe the exposure time according to the manufacturer's instructions). Wipe disinfection is generally preferable to spray disinfection. After the disinfectant has taken effect, dry the rollator with a clean, dry cloth.

 **ATTENTION:** Do not use disinfectants containing chlorine or phenol, or vinegar-based cleaning agents.

8.3 Hygiene plan before each patient change

Before each re-use or change of patient, the rollator must be prepared as follows:

1. **Cleaning** (Follow the instructions in Chapter 8.1)
2. **Disinfection** (Follow the instructions in chapter 8.2)

ATTENTION The disinfection of the rollator during a change of patient may only be carried out by a specialist.

8.4 Reuse

The product is suitable for reuse if it has been cleaned and disinfected in accordance with the instructions in chapter 8.3. In addition, it must be inspected by a specialist dealer for damage and then released for reuse. Have your rollator serviced by your specialist dealer at regular intervals. They will detect any wear and replace components if necessary. If you notice any faults during the regular inspection of your rollator's braking force, please contact your specialist dealer immediately and in confidence!

9. MAINTENANCE

As a user of the rollator, you are always the first to notice any damage. Check the condition of the rollator regularly. If you notice any defects during your inspection, contact your specialist dealer immediately. The rollator must not be used without a professional repair. We recommend that you have your rollator checked by a qualified dealer at least once a year.

9.1 Maintenance table

What needs to be checked?	Check for function	 Notes
General condition	Check all frame parts, including the welds, for deformation, damage, wear and corrosion.	Defective product parts must be replaced before reuse.
Fold-protection	Check that the folding lock engages properly (see chapter 5.2)	If the folding mechanism is not working properly, it must be repaired or replaced.
Fasteners on tight fit	Check, for example, whether the locking lever with the screw is properly fixing the push-handle. (See chapter 5.4)	If any loosened or missing screws, nuts, rivets, etc. are found, they must be tightened, added or replaced.
Functional test of the brake system	Apply the service brake as far as it will go. The wheels should now no longer turn when pushed forcefully. Apply the parking brake by pressing the brake lever down until it engages. The wheels should now no longer turn when pushed forcefully.	If a brake is faulty, the brake play, brake cable, brake shoes and wheels must be checked and, if necessary, the defective part replaced.
Roller profile and ball bearing	Check the rollers for heavy wear/defects and check the ball bearing for damage.	If there is wear, defects or cracks, the rollers or ball bearing must be replaced.
Wheels (PU) Function	The castors must roll straight, easily and quietly and must not wobble. Check that the rollator rolls straight ahead.	If damaged or heavily worn, the rollers must be replaced.
Push handles	The handgrips must be firmly attached, the push handles must be at the same height and must not show any signs of damage or bending. The locking screws must lock securely.	Serious defects to handles/push handles or locking screws must be rectified.
Cleaning	Check for dirt	The entire rollator should be cleaned depending on how dirty it is, but at least once a month (see chapter 8. Care and hygiene).

GB

9.2 Parts to be serviced:

Frame parts, wheels, brake cables, brake lever, brake bolts, push handles, seat, folding lock, storage tray, luggage basket, locking lever, bearings, knobs, screws and nuts.

9.3 Check the following before every journey:

- Do the brakes work properly?
- Do the castors work properly? Test whether the rollator rolls straight ahead.
- Are all removable parts securely attached (basket, tray)?
- Is there any visible damage (e.g. to the frame, wheels, brake cables, brake levers)?
- Is there any material damage such as cracks?

9.4 Check the following once a month:

- Check the tyre wear every four weeks. Do all wheels run smoothly when idling? Make sure that no dirt has accumulated in the wheel hubs. The service life of the wheels depends on use. Defective wheels with wear must be replaced.
- All moving parts must be checked. Does the folding mechanism work smoothly and silently? Squeaking noises should be checked by a specialist dealer.
- Check that the screws and nuts on the rollator are tight.

10. STORAGE

Store the rollator in a dry place, protected from light and dust. Do not store near heat sources. Protect the rollator from dirt or contamination after preparation or if it is not used for a longer period of time.

11. WARRANTY

We provide a warranty for this product in accordance with European statutory provisions. The first 12 months are the warranty period. This warranty covers manufacturing and assembly defects. We are not liable for wear and tear or careless and improper use. The warranty is void if the product is modified or repaired by unauthorised dealers or persons. It does not include costs for troubleshooting, freight, loss of use or earnings, for necessary travel and similar costs. Also not covered by the warranty are damages or injuries resulting from improper use, misuse or failure to follow the operating instructions. It is not permitted to make technical changes to the product.

12. ATTENTION!

Disposal

This device must NOT be disposed of with household waste. For proper disposal, please contact the public collection points in your community. For details on the location of such a collection point and on any daily/monthly/annual quantity restrictions that may apply, as well as on any costs that may be incurred for collection, please refer to the information provided by the respective municipality.

GB

Use stop

In case of defects or uncertainties, the rollator must not be used. In such cases, please contact your dealer or the importer.

Additional equipment

Use only original accessories!

Seek advice from a health professional

If you are unsure about how to use the rollator or have questions about its suitability and ergonomics, you should also consult a healthcare professional for advice.

Reporting of serious incidents

Should a serious incident occur in connection with this medical device, please contact the manufacturer of this product, the importer and the authorised representative.

No liability is accepted for consequential damage.
Subject to technical modifications and errors!

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Proteno GmbH